



Biological Teeth Whitening

# Biological Teeth Whitening

## ProOFFICE and ProHOME



## Alkaline-HP & Nano-HAp Protocol

**BioWhiten**

The BIOWHITEN ProOffice ULTIMATE system has less than a 5% sensitivity level, and many dentists indicate a sensitivity level close to zero.

The ProOffice ULTIMATE protocol does NOT require the use of desensitization products or additional measures.

No light source needed.



PERSONALIZED WHITENING TREATMENT PLAN,  
EFFECTIVE PRODUCTS AGAINST TOOTH SENSITIVITY...

ULTIMATE NANO WHITENING





The system creating bidirectional whitening that contains both low-level Alkaline Hydrogen Peroxide (a-HP) and Nano-Hydroxyapatite (n-HAp) crystals that are compatible with the crystal structure of the tooth and giving the tooth its natural color is called biological teeth whitening.

With such a system, in addition to an effective tooth whitening application, simultaneous mineral supplementation (remineralization) can be performed. Office, Home and Home + Office Combined systems used in the same day allow a flexible application without any side effects and sensitivity.

VIRTUALLY ZERO SENSITIVITY, THE FIRST AND ONLY  
WHITENING PRODUCT THAT CAN BE APPLIED TO SENSITIVE TEETH...



ULTIMATE NANO WHITENING



# BioWhiten

## In-office Whitening

Hydrogen Peroxide & Nano-Hydroxyapatite

## Biological Tooth Whitening

BIOWHITEN ProOffice POWERHP gives the best possible whitening result in 3 x15 minutes treatment time.



### System

- POWERHP in 2.5 ml dual-chamber syringes
- GINGIVA BARRIER in 2.2 ml syringes
- There is no need for desensitizing and remineralization agent since It contains best biocompatible Nano-Hydroxyapatite material

### Available concentrations

- 40% HP, 25% HP, 18% HP

### Available Packages

- 2 x 2,5ml POWERHP Syringe and 1x2.2ml Barrier - 2 Patients Kit
- 4 x 2,5ml POWERHP Syringe and 2x2.2ml Barrier - 4 Patients Kit

### Accessories

- MicroBrush for adjustment and mixing
- 2 x Auto Mix Tips per syringe
- 2 x Barrier Tips per syringe



# BioWhiten

## In-office Whitening

Alkaline Hydrogen Peroxide & Nano-Hydroxyapatite

## Biological Tooth Whitening

BIOWHITEN ProOffice ULTIMATE gives the best possible whitening result with lowest possible sensitivity level and maximum remineralization.

Suitable for Sensitive Teeth!



### System

- 15ml Hydrogen Peroxide Bottle & 15ml Nano-Hydroxyapatite Bottle
- GINGIVA BARRIER in 2.2 ml-syringes
- There is no need for desensitizing and remineralization agent  
It contains maximum biocompatible Nano-Hydroxyapatite material

### Available concentrations

- 6% HP, Scalable 12% & 18 % HP

### Available Packages

- 2x 15ml Bottles and 2x 2.2ml Barrier - Upto 4 Patients Kit

### Accessories

- MicroBrush for adjustment and mixing
- Mixing Cups and 5x5cm Sponges
- 2 x Barrier Tips per syringe

ULTIMATE NANO WHITENING



# BioWhiten

## Home Whitening

Alkaline Hydrogen Peroxide & Nano-Hydroxyapatite

### Biological Tooth Whitening

BIOWHITEN ProHome 6% HP  
Day time use, maximum 1 hour per day.  
Shortest application time,  
Zero sensitivity level and maximum  
remineralization.



#### System

- 2.5ml dual-chamber syringes
- There is no need for desensitizing and remineralization agent  
It contains maximum biocompatible Nano-Hydroxyapatite material

#### Available concentrations

- 6% HP,

#### Available Packages

- 4 x 2.5ml Syringes - Upto 14 Days application time
- 20 x 2.5ml Syringes - Bulk Package

#### Accessories





- 1x Auto Mixing Tips per Syringe

ULTIMATE NANO WHITENING



# PREDICTABLE AND PROFESSIONAL WHITENING SYSTEMS, SUITABLE FOR MOST DIFFICULT CASES...



	Method	Application time per session	Number of applications
	<b>ProOffice Ultimate</b> The fast reaction in-office whitening ALCALINE HP and NHAP Mix, Suitable for Sensitive Teeth NO LIGHT NEEDED. It can be used with or without light source with max. 39-40 C heat level.	6% HP & 12-18% Scalable System 5 x 10 Min (max. 8 x 10 Min)	1 session - 50 minutes or 80 min.  2nd sessions may be required for very exceptional cases
	<b>ProOffice PowerHP</b> Effective and fast method in-office tooth whitening system with low sensitivity level. NO LIGHT NEEDED. It can be used with or without light source with max. 39-40 C heat level.	- 40% HP & n-HAP , 3 x 15 Min - 25% HP & n-HAP System 3 x 15 Min or 4 x 10 Min - 18% HP & n-HAP System 5 x 10 Min	1 session - 45 Minutes  Dentist can decide to apply additional 1 x 15min or 2 x 10min.  1 session - 50 Minutes Dentist can decide to apply additional 2 x 10min or 3 x 10min.
	<b>ProHome - Home Whitening</b> Home teeth whitening treatment to be used in home or office.  WITH CUSTOM-MADE TRAYS	6% HP & n-HAP System Daytime use, 45 Min - 60 Min per day	4-14 days  It can be combined with office whitening and same day can be applied.  Dentist can decide exact duration
	<b>ProBar Barrier</b> Light-curing, GINGIVA PROTECTOR	2.2ml syringe 10-15 seconds light curing time	1 syringe can be used for 2 treatments (both upper&lower jaws)

# Biological Teeth Whitening

Biodent Medical Ltd.


www.biowhiten.com    info@biowhiten.com    +90 216 545 5 595



## BioWhiten

BioWhiten, the most effective and latest nano technology teeth whitening system, offers an effective whitening against different tooth discolorations, from the simplest to the most complex cases.

In addition to normal coloration, it allows you to make all kinds of teeth whitening, including tetracycline cases at the most affordable cost.

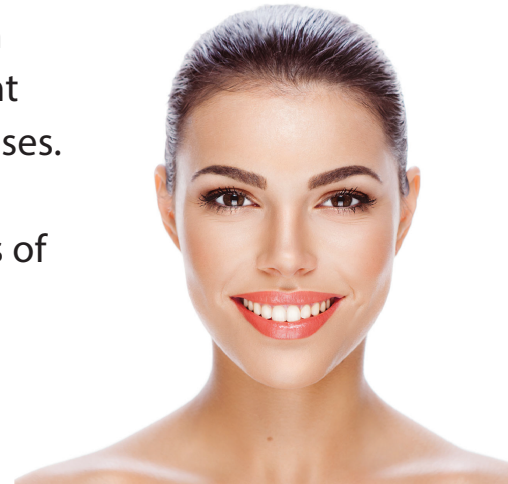

Teeth Whitening NO LONGER has to be associated with sensitivity and demineralization on the teeth.

ULTIMATE NANO WHITENING